

Use of Kneehab[®] XP

Conductive Gel Pad Placement:

The inside of the garment has been printed with conductive gel pad placement guides, showing you exactly where to place the 4 conductive gel pads. The locations will vary depending on your size (A/a and B/b).



1. Lay out the conductive garment on a flat surface with the white markings facing you.



3. Place the gel pads, grid pattern side down, firmly on the garment within area determined during initial fitting.



2. Remove the clear plastic liners from the patterned side of the 4 conductive gel pads.



4. Remove the plastic liners from the gel pads. Do not throw these away as you will need them later for storing the Kneehab XP.

Conductive Gel Pad Care:

Your conductive gel pads will last longer with proper care:

- + After use, leave gel pads on the Kneehab and place the plastic liners back on the pads.
- + Place garment in storage bag.
- + To improve adhesive strength, re-hydrate gel pad with a few drops of water and massage gently into gel. With proper care, as per manual, gel pads have an estimated usage of 20 sessions.

See Kneehab XP manual for complete operating instructions.



Warning

- + Use only an approved neurotech device.
- + Before applying the gel pads to the Kneehab, carefully read the safety advice and user instructions.
- + Use only approved neurotech gel pads.
- + Do not use when driving.
- + Do not apply pads to areas of broken or inflamed skin.
- + Clean skin thoroughly before application.
- + Gel pads are single-person use only.

neurotech[®]
embrace life.



Kneehab[®] XP

Manufactured for:

Theragen Inc.
11220 Assett Loop, Suite 101
Manassas, VA 20109, USA

Tel: 1-888-980-1197
Website: www.kneehab.com
Email: patientcare@neurotech.us

Part number: 2481-3300T

Issue date: 10/19

Rev.2

Type: 613/614/615/616

Storage temperature: 32 - 130°F (0 - 55°C)

Storage humidity: 10 - 90% RH

U.S. Patent Number 6,944,503
Copyright © Theragen Inc. All rights reserved.

Caution: In the United States of America federal law restricts the device to sale or use by, or on the order of a physician or other practitioner licensed by the laws of the state in which he/she practices.

neurotech[®] and Kneehab[®] are registered trademarks of Theragen Inc.



Read
instructions



Keep dry

Gel pads made in U.S.A.